

Cleaning and Replacing Guidelines

Regularly cleaning and replacing the accessories is essential for hygiene and ensuring that your therapy remains as **clean; comfortable,** and effective as possible.

Accessory Component	Cleaning Schedule	Cleaning Method	Replacing Schedule
Nasal / Pillows Cushion ¹	Daily	A. Wash in a solution of warm water and mild dishwashing liquid or soapy water	Twice per month
Full Face Cushion ¹	Daily		Every 3 months
Mask Frame ¹	Daily	B. Rinse thoroughly with clean water	Every 3 months
Mask Headgear ²	Weekly		Every 6 months
Air Tubing	Weekly	C. Air dry away from direct sunlight	Every 3 months
Water Chamber of Humidifier	Weekly	C. Air dry away from direct sunlight	Every 6 months
Reusable Air Filter	Every 2 weeks	A. Wash with running tap water	Every 6 months
Optional Disposable Ultra-Fine Filter	Every 2 weeks	C. Air dry away from direct sunlight	Every 3 months
Optional Ultra-Fine Filter	NOT washable or reusable		Every 2 weeks

• ¹Refers to soft material)

• ²Refers to soft fabric parts)

• (Refers to soft fabric parts)